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1
Separate the edges of the mask to fully open it

2
Bend the nose wire to form a gentle curve. The nose wire represents the top of the mask

3
Hold the mask upside down to expose the two straps

4
Using your index fingers and thumbs, separate the two straps

5
While holding the straps, cup the mask under your chin

6
Pull the straps up and over your head

7
Place and position the lower strap at the base of your neck (under your ears)

8
Place the upper strap on the crown of your head so that it runs just above the top of your ears

9
Gently press the nose wire down across the bridge of your nose until it fits snugly

10
Continue to adjust the mask and edges until you feel you have achieved a good and comfortable fit

FIT CHECK

Gently inhale. When you breathe in the mask should draw in slightly toward the face and collapse

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12

Gently exhale. The mask should fill up with air. It is important at this stage that there is no air leakage around edges of mask.

Follow these directions each time the mask is worn. The masks will not provide the intended level of protection unless worn correctly so it is important that you achieve a good fit as detailed above. Men who are clean shaven will achieve the best facial seal. The brand of P2 or N95 masks may vary slightly so always follow the manufacturer's instructions and warnings.

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அங்கீகரித்து, வெளியிட்டவர்: விக்ரோரிய அரசாங்கம், 1 Treasury Place, Melbourne. © விக்ரோரிய மாநிலம், Department of Health and Human Services, பெப்ரவரி 2016.